

SEE THINGS DIFFERENTLY

EVERYWHERE YOU LOOK



**When you invest in your vision,
you're investing in your future.**

Redefine your vision with
Clareon® PanOptix®, the #1 trifocal
lens with over 3 million implants
worldwide.^{10,11} Enjoy a full range of
vision and exceptional clarity, and
reduce your need for glasses.^{1,2}



START YOUR JOURNEY WITHOUT CATARACTS

Cataracts form gradually as our eyes get older, and 9 in 10 Americans develop them by the time they're 65.³ The good news is that cataract surgery is one of the safest, most common, and most effective procedures performed today.⁴

1. Cataract Removal

Through a tiny incision, the surgeon removes your clouded lens.

2. Lens Insertion

The surgeon replaces your original lens with the **Clareon® PanOptix®** Lens.

3. Vision Renewed

The **Clareon® PanOptix®** Lens allows you to see clearly near to far.¹

A LENS FOR YOUR LIFE

Unlike a monofocal intraocular lens, the **Clareon® PanOptix®** lens gives you a full range of vision, so that you can immerse in a full range of activities — all while relying less on glasses.^{1,2}



Enhance your vision for clear, complete focus^{1,5}



See effortlessly, from near to far¹




Discover bright, vivid colors⁶




Reduce your need to wear glasses^{1,2}

SEE THINGS DIFFERENTLY FREE FROM CATARACTS WITH THE CLAREON® PA

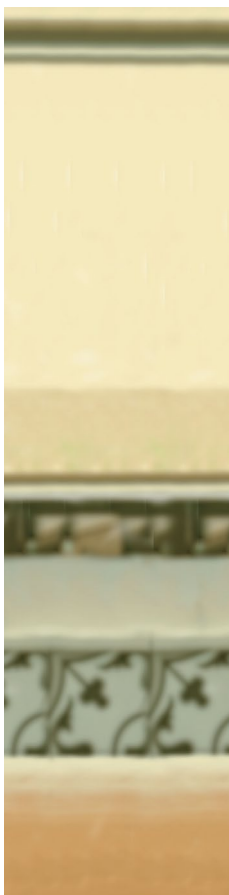


Cataracts
Cloudy

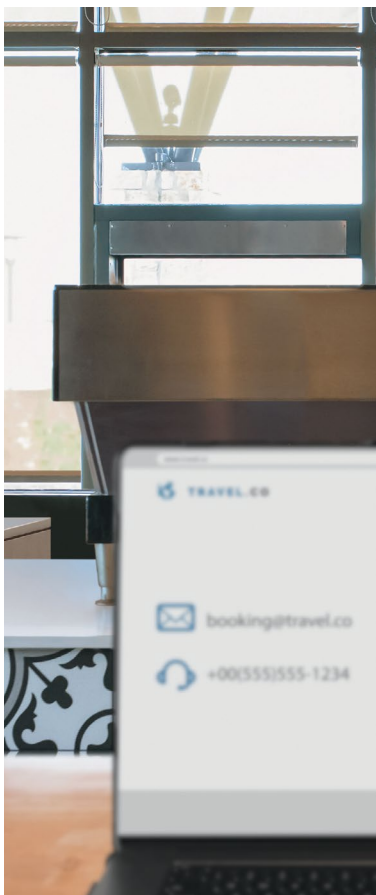

Monofocal and **Monofocal Toric* Lens** provides clear distance vision.^{5,8,9} You may need glasses for clear near and intermediate vision.⁷

*Provides astigmatism correction⁷

M ANOPTIX® LENS

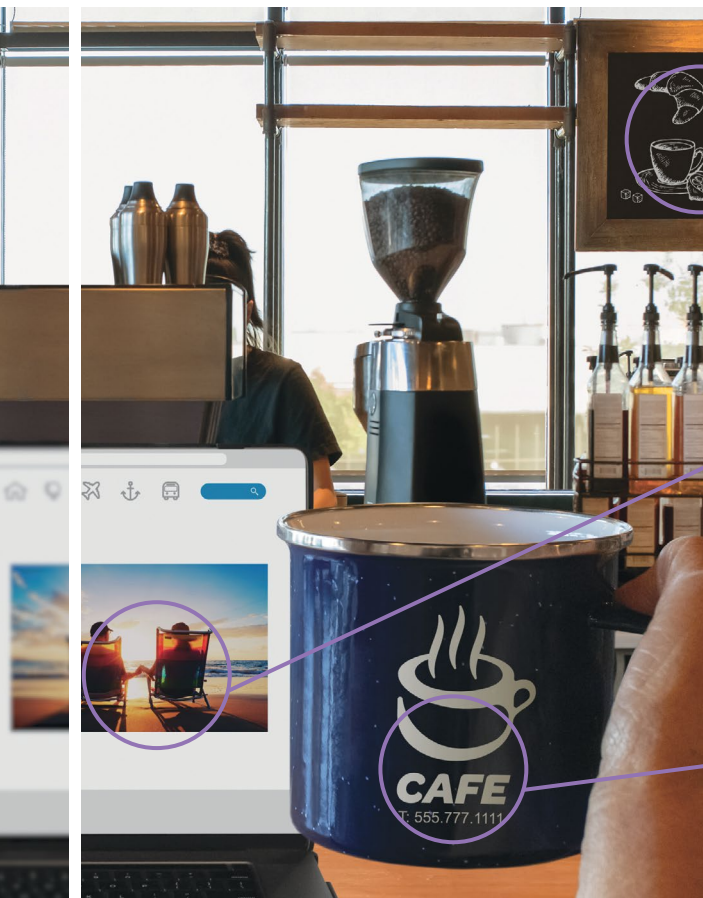

Clareon® PanOptix® Lens

Provides distance, intermediate, and near vision.¹



CLEAR VISION FAR AWAY

- Spot a familiar face
- Drive a car
- Watch TV

ENHANCED VISION AT ARM'S LENGTH

- Use a computer
- Read a menu
- See in a mirror

EXCELLENT VISION UP CLOSE

- Read a book
- Take up crafting
- Use a cell phone



Clareon® PanOptix®
Trifocal IOL

PATIENTS LOVE PANOPTIX®

In a clinical study, 129 patients were asked about their experience with the **PanOptix®** Lens.

99.2%*

of patients **would choose the same lens again.**¹

98%*

of patients **would recommend it to family and friends.**¹

Other studies reveal that

91.6%*

of patients are **free from glasses.**²



**FIND OUT MORE AT
MYCATARACTS.COM**

*Study conducted on Acrysof® PanOptix® patients. Acrysof® PanOptix® and Clareon® PanOptix® are optically equivalent.

IMPORTANT PRODUCT INFORMATION - CLAREON® FAMILY OF IOLS

CAUTION: Restricted by law to sale by or on the order of a physician.

DESCRIPTION: The **Clareon® Family of Lenses** are artificial lenses implanted in the eye of adult patients following cataract surgery. The **Clareon® Aspheric Hydrophobic Acrylic IOLs** are designed to allow for clear distance vision. However, you will likely still need glasses for reading and for distance vision particularly if you already have astigmatism. The **Clareon® PanOptix® Trifocal Hydrophobic IOL** is a type of multifocal lens (sometimes called “presbyopia-correcting IOL”) designed to allow for clear distance, intermediate, and near vision with the potential to be more independent of the need to use glasses for daily tasks. The **Clareon® Vivify® Extended Vision Hydrophobic Posterior Chamber IOL** provides clear distance vision, and better intermediate and some near vision compared to a monofocal IOL. The **Clareon® Aspheric Toric, Clareon® PanOptix® Toric, and Clareon® Vivify® Toric IOLs** are also designed to correct pre-existing corneal astigmatism, which is the inability of the eye to focus clearly at any distance because of difference curvatures on the cornea, and provide distance vision.

WARNINGS / PRECAUTIONS: You may experience and need to contact your eye doctor immediately if you have any of the following symptoms after cataract surgery: itching, redness, watering of your eye, sensitivity to light. The safety and effectiveness of these IOLs have not been established in patients with eye conditions, such as an increase in eye pressure (glaucoma) or complications of diabetes in the eye (diabetic retinopathy). As with any surgical procedure, there are risks involved. These risks may include but are not limited to infection, damage to the lining of the cornea, the retinal layer which lines the inside back wall of your eye may become separated from the tissue next to it (retinal detachment), inflammation or swelling inside or outside the eye, damage to the iris (the colored diaphragm around the pupil), or an increase in eye pressure that cannot be controlled by medicine and secondary surgical procedure. There is a possibility that the IOL may be placed incorrectly or could move within the eye. This may result in less improvement or a reduction in vision, or it may cause visual symptoms. The **Clareon® Aspheric Toric, Clareon® PanOptix® Toric, and Clareon® Vivify® Toric IOLs** correct astigmatism only when placed in the correct position in the eye. There is a possibility that these Toric IOLs could be placed incorrectly or could move within the eye. This may result in less improvement or a reduction in vision because your astigmatism has not been fully corrected, or it may cause visual symptoms. With the **Clareon® PanOptix® and Clareon® Vivify® IOLs**, there may be a loss of sharpness of your vision that may become worse in dim light or in foggy conditions. There is also a possibility that you may have some visual effects such as rings or circles (halos) around lights at night. You may also have trouble seeing street signs due to bright lights or glare from oncoming headlights.

ATTENTION: As with any surgical procedure, there are risks involved. Prior to surgery, ask your eye doctor to provide you with the Patient Information Brochure for the lens to be implanted. This Brochure which will inform you of the risks and benefits associated with the IOL. Discuss any questions about possible risks and benefits associated with your eye doctor.

REFERENCES: 1. Clareon® PanOptix® Trifocal Hydrophobic Acrylic IOL Model CNWTT0 2021 Directions for Use. 2. Zhu D, Ren S, Mills K, Hull J, Dhariwal M. Rate of Complete Spectacle Independence with a Trifocal Intraocular Lens: A Systematic Literature Review and Meta-Analysis. *Ophthalmol Ther.* 2023;12(2):1157-1171. 3. Cataract. Kellogg Eye Center website. <https://www.umkelloggeye.org/conditiontreatments/cataract>. Accessed April 13, 2023. 4. Davis G. The Evolution of Cataract Surgery. *Mo Med.* 2016;113(1):58-62. 5. Lehmann R, Maxwell A, Lubeck DM, Fong R, Walters TR, Fakadej A. Effectiveness and Safety of the Clareon Monofocal Intraocular Lens: Outcomes from a 12-Month Single-Arm Clinical Study in a Large Sample. *Clin Ophthalmol.* 2021;15:1647-1657. Published 2021 Apr 20. 6. Groessl EJ, Liu L, Sklar M, Tally SR, Kaplan RM, Ganiats TG. Measuring the impact of cataract surgery on generic and vision-specific quality of life. *Qual Life Res.* 2013;22(6):1405-1414. 7. Clareon® Toric Directions for Use. 8. Alcon data on file, 2023. 9. Werner L, Thatthamla I, Ong M, et al. Evaluation of clarity characteristics in a new hydrophobic acrylic IOL. *J Cataract Refract Surg.* 2019;45:1490-1497. 10. Alcon data on file, 2024 REF-24102. 11. MarketScope LLC. 2023 IOL Market Report: Global Analysis for 2022 to 2028. St. Louis, MO: MarketScope LLC; 2023. 12. Alcon, Data on file 2024.



**FIND OUT MORE AT
MYCATARACTS.COM**

Alcon