

IMPORTANT PRODUCT INFORMATION - CLAREON® FAMILY OF IOLS

CAUTION: Restricted by law to sale by or on the order of a physician.

DESCRIPTION: The **Clareon® Family of Lenses** are artificial lenses implanted in the eye of adult patients following cataract surgery. The **Clareon® Aspheric Hydrophobic Acrylic IOLs** are designed to allow for clear distance vision. However, you will likely still need glasses for reading and for distance vision particularly if you already have astigmatism. The **Clareon® PanOptix® Trifocal Hydrophobic IOL** is a type of multifocal lens (sometimes called “presbyopia-correcting IOL”) designed to allow for clear distance, intermediate, and near vision with the potential to be more independent of the need to use glasses for daily tasks. The **Clareon® Vivity® Extended Vision Hydrophobic Posterior Chamber IOL** provides clear distance vision, and better intermediate and some near vision compared to a monofocal IOL. The **Clareon® Aspheric Toric, Clareon® PanOptix® Toric, and Clareon® Vivity® Toric IOLs** are also designed to correct pre-existing corneal astigmatism, which is the inability of the eye to focus clearly at any distance because of difference curvatures on the cornea, and provide distance vision.

WARNINGS / PRECAUTIONS: You may experience and need to contact your eye doctor immediately if you have any of the following symptoms after cataract surgery: itching, redness, watering of your eye, sensitivity to light. The safety and effectiveness of these IOLs have not been established in patients with eye conditions, such as an increase in eye pressure (glaucoma) or complications of diabetes in the eye (diabetic retinopathy). As with any surgical procedure, there are risks involved. These risks may include but are not limited to infection, damage to the lining of the cornea, the retinal layer which lines the inside back wall of your eye may become separated from the tissue next to it (retinal detachment), inflammation or swelling inside or outside the eye, damage to the iris (the colored diaphragm around the pupil), or an increase in eye pressure that cannot be controlled by medicine and secondary surgical procedure. There is a possibility that the IOL may be placed incorrectly or could move within the eye. This may result in less improvement or a reduction in vision, or it may cause visual symptoms. The **Clareon® Aspheric Toric, Clareon® PanOptix® Toric, and Clareon® Vivity® Toric IOLs** correct astigmatism only when placed in the correct position in the eye. There is a possibility that these Toric IOLs could be placed incorrectly or could move within the eye. This may result in less improvement or a reduction in vision because your astigmatism has not been fully corrected, or it may cause visual symptoms. With the **Clareon® PanOptix® and Clareon® Vivity® IOLs**, there may be a loss of sharpness of your vision that may become worse in dim light or in foggy conditions. There is also a possibility that you may have some visual effects such as rings or circles (halos) around lights at night. You may also have trouble seeing street signs due to bright lights or glare from oncoming headlights.

ATTENTION: As with any surgical procedure, there are risks involved. Prior to surgery, ask your eye doctor to provide you with the Patient Information Brochure for the lens to be implanted. This Brochure which will inform you of the risks and benefits associated with the IOL. Discuss any questions about possible risks and benefits associated with your eye doctor.

References: 1. Cataract. Kellogg Eye Center website. <https://www.umkelloggeye.org/conditions-treatments/cataract>. Accessed April 13, 2023. 2. Davis G. The Evolution of Cataract Surgery. *Mo Med*. 2016;113(1):58-62. 3. Clareon® Vivity® Extended Vision Hydrophobic IOL (CNWET0) Directions for Use – USA. 4. Clareon® Toric Directions for Use. 5. Alcon data on file, 2023. 6. Lehmann R, Maxwell A, Lubeck DM, Fong R, Walters TR, Fakadej A. Effectiveness and Safety of the Clareon Monofocal Intraocular Lens: Outcomes from a 12-Month Single-Arm Clinical Study in a Large Sample. *Clin Ophthalmol*. 2021;15:1647-1657. Published 2021 Apr 20. 7. Werner L, Thatthamla I, Ong M, et al. Evaluation of clarity characteristics in a new hydrophobic acrylic IOL. *J Cataract Refract Surg*. 2019;45:1490-1497. 8. Alcon data on file, 2019.



**FIND OUT MORE AT
MYCATARACTS.COM**

Alcon

SEE THINGS DIFFERENTLY

GET THE WHOLE PICTURE



**When you invest in your vision,
you're investing in your future.**

Cataracts don't have to hold you back from doing the things you love. See what makes your life remarkable with the **Clareon® Vivity®** Lens.

Alcon



START YOUR JOURNEY WITHOUT CATARACTS

Cataracts form gradually as our eyes get older, and 9 in 10 Americans develop them by the time they're 65.¹ The good news is that cataract surgery is one of the safest, most common, and most effective procedures performed today.²

1. Cataract Removal

Through a tiny incision, the surgeon removes your clouded lens.

2. Lens Insertion

The surgeon replaces your original lens with the **Clareon® Vivity®** Lens.


3. Vision Renewed

The **Clareon® Vivity®** Lens enhances your vision so you'll be free of cataracts.

CLARITY FOR EVERYDAY TASKS

The way you use your vision shapes the way you live your life. The **Clareon® Vivity®** Lens shines a light on what's important to you every day.

In a clinical study with the **Vivity®** Lens:

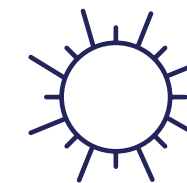

94% reported good or very good vision far away without glasses in bright light.³




92% reported good or very good vision at arm's length without glasses in bright light.³

Unlike a monofocal intraocular lens, the **Clareon® Vivity®** Lens allows you to see clearly at the crucial far and intermediate distances with functional near vision, so that you can immerse yourself in a full range of activities—all while relying less on glasses.³



Cataracts
Cloudy

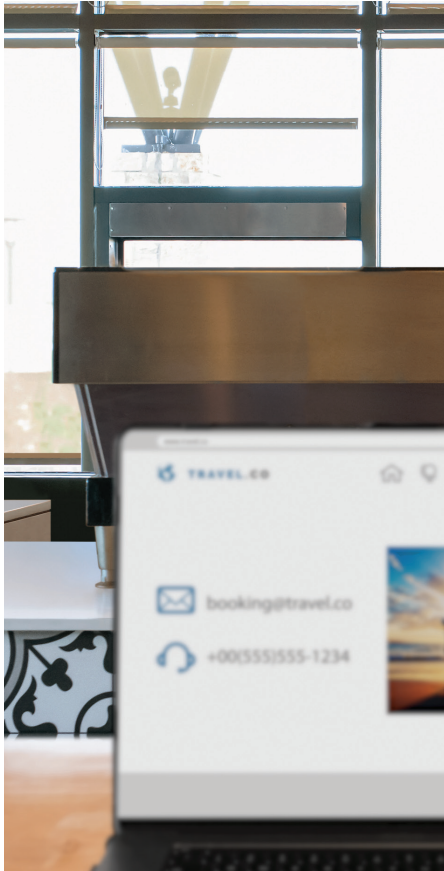

Monofocal and Monofocal Toric* Lens provides clear distance vision.⁵⁻⁷ You may need glasses for clear near and intermediate vision.⁴

*Provides astigmatism correction⁴



Clareon® Vivity® Lens
Provides distance, intermediate, and functional near vision.³

CLEAR VISION FAR AWAY

- Spot a familiar face
- Drive a car
- Watch TV

CLEAR VISION AT ARM'S LENGTH

- Use a computer
- Read a menu
- See in a mirror

FUNCTIONAL VISION UP CLOSE

- Tend a garden
- Play games
- Take a picture

PATIENTS LOVE VIVITY®

In a clinical study, 106 patients were asked about their experience with the **Vivity® Lens**.

OVER

90%

reported that they were satisfied with their vision.⁸

94%

of patients reported rarely or never needing eyeglasses or contacts at a distance.⁸



FIND OUT MORE AT
MYCATARACTS.COM